

Green Card T Air Removal & **STEAM** Penetration Test

BD 120

The Green Card T is for daily monitoring of small pre-vacuum type B steam sterilizers operating at 134°C. The card is specially packaged and designed to meet EN 867-5 requirements.

The Green Card T is tested and verified to have equivalent and approved performance at both EN 867-5 test cycles and Bowie-Dick cycles for type B pre-vacuum table top sterilizers.



Key Features

- Complete color change - 3.5 minutes at 134°C
- Sterilization pouch protects the indicator from excessively wet conditions
- Lead free and no toxic heavy metals

Performance Characteristics

If the sterilizer is functioning properly, the indicator graphic will show a color change from purple to a uniform green throughout the entire indicator image. When excessive residual air or non-condensable gases are present (major failure), the indicator will not be a uniform green color but remain purple in the center or a shade of gray.

Physical Characteristics

- Dimensions: L 86mm x W 86mm

Storage

The storage conditions are printed on the product packaging. Protect from moisture and humidity. Store in resealable bag provided.

Shelf Life

24 months from date of manufacture. The expiry date is printed on the product packaging.

Standards Classification

EN 867-5

SteriTec Part No.

BD 120, Green Card T
Air Removal & Steam
Penetration Test

Packaging

15 bags of 15
tests + 1 ring

FTP-168 Rev. H