

Getinge Clean

Acid Detergent

Organic acid, instrument cleaner detergent.



Application

For use in mechanical cleaning equipment, cart washers, instrument washers and washer-disinfectors.

Key Features

- Has a strong dissolving action against minerals and scale
- Can be used in laboratories, hospitals and other institutions; on instruments, utensils, glassware and animal cages
- Removes detergent residuals from washer-decontaminators

Usage Information

Acid Detergent should be used at a dose of 0.2–1.0 oz/ gal (2–8 ml/L) depending on water quality. Always follow manufacturer's directions for the particular automatic washer being used. If you have additional questions, consult your Getinge representative.

- Do not add bleach, detergents, or any other chemical to this product. Added chemical may destroy the effectiveness of this cleaner.

NOTE: Some plastics and rubbers are not compatible with detergents or high water temperatures. Check with manufacturer for suitability.

Quality Statement: Confidence in Getinge is the most important quality criteria. This is the hallmark of all our external and internal commitments, activities and products. Products and services supplied by Getinge conform to the agreed terms and expectations. The achievement of these quality goals is the basis for continued competitive and successful enterprise.

Physical and Chemical Properties

- Clear, colorless, thin liquid
- Complete water solubility
- Characteristic fragrance
- pH value: 1.6–2.6

NOTICE: For institutional and industrial use only. Not for home use. Refer to Safety Data Sheet (SDS) for more information regarding safety, handling and storage and waste disposal methods.

Ordering Information

Getinge Clean	Order Number
4 x 1 Gallon	61301605278
5 Gallons	61301600042
15 Gallons	61301600043
30 Gallons	61301602229

Getinge is a registered trademark of Getinge AB, its subsidiaries, or affiliates in the United States or other countries • Copyright 2018 Getinge AB or its subsidiaries or affiliates • All rights reserved. • MCV00044162 REV B

Manufacturer: Intercon Chemical Co. 1100 Central Industrial Dr., St. Louis, MO 63110
Sales Office: Getinge USA Sales, LLC, 1777 East Henrietta Road, Rochester, NY 14623

www.getinge.com

GETINGE 