

Getinge Clean

Instrument Lubricant Plus

Instrument Lubricant is a steam and ethylene oxide permeable instrument lubricant.



Application

Instrument Lubricant Plus is a formulation designed for immersion application and for use in instrument washers and washer-disinfectors. When used on a regular basis, Instrument Lubricant Plus reduces cost of instrument repair and replacement.

Key Features

- Deposits a thin film of lubricant to prevent rusting, discoloration and corrosion
- Steam and EtO permeable, does not interfere with sterilization
- Extends the life of an instrument and ensures proper working of hinges, pins and joints
- Easier cleaning; no rinsing or wiping required; after cleaning, immerse instruments into the diluted solution; proceed to sterilization

Quality Statement: Confidence in Getinge is the most important quality criteria. This is the hallmark of all our external and internal commitments, activities and products. Products and services supplied by Getinge conform to the agreed terms and expectations. The achievement of these quality goals is the basis for continued competitive and successful enterprise.

Usage Information

For Immersion Application:

1. Shake well before using.
2. Mix 1:6 with deionized water.
3. Immerse clean instruments (opened) into the solution for 30 to 45 seconds.
4. Remove instruments and allow them to drain. Rinsing and wiping of instruments is not necessary.
5. The lubricated instruments are ready for sterilization.

IMPORTANT: Open bath diluted solutions should be discarded after each day.

For Automatic Washer Application:

Set up lubricant delivery system to inject Instrument Lubricant into the final distilled water rinse of the washer cleaning process at a rate of 0.1 oz/gal (0.8ml/L) to 0.25 oz/gal (2 ml/L) of rinse water used. If you have additional questions, consult your Getinge representative.

- Do not add bleach, detergents, or any other chemical to this product. Added chemical may destroy the effectiveness of this cleaner.

NOTE: Some plastics and rubbers are not compatible with detergents or high water temperatures. Check with manufacturer for suitability.

Physical and Chemical Properties

- White liquid
- Mild fragrance
- pH value: 7.0–8.0

NOTICE: For institutional and industrial use only. Not for home use. Refer to Safety Data Sheet (SDS) for more information regarding safety, handling, storage and waste disposal methods.

Ordering Information

Getinge Clean	Order Number
4 x 1 Gallon	61301605247
2.5 Gallons	61301606790
5 Gallons	61301605248
15 Gallons	61301605249

Getinge is a registered trademark of Getinge AB, its subsidiaries, or affiliates in the United States or other countries • Copyright 2018 Getinge AB or its subsidiaries or affiliates • All rights reserved. • MCV00044175 REV B

Manufacturer: Intercon Chemical Co. 1100 Central Industrial Dr., St. Louis, MO 63110
Sales Office: Getinge USA Sales, LLC, 1777 East Henrietta Road, Rochester, NY 14623

www.getinge.com

GETINGE 