



Product Specification
**Getinge da Vinci Rack
Accessory**

Getinge da Vinci Rack Accessory

Product Details

Application

The Getinge da Vinci Rack is tailor-made for surgical instruments from Intuitive Surgical Inc used in the da Vinci Surgical System for robotic-assisted surgery (5 and 8 mm EndoWrist instruments, Stapler and Single site). When used in combination with the Intuitive Surgical's reprocessing instructions, it assures a reliable thermal disinfection of EndoWrist instruments.

The da Vinci Rack used with the S-8668, S-8668T or 88-series washer-disinfectors with integrated wash-carts, enables thorough rinsing and thermal disinfection of da Vinci EndoWrist instruments, both internally and externally.

Intended use, tests and verifications

The Getinge da Vinci Rack has been tested and verified in cooperation with Intuitive Surgical Inc, for an optimal and reliable disinfection of EndoWrist instruments.

Note! Thermal disinfection is not a substitute for reprocessing. The manufacturers Instruction For Use (IFU) needs to be followed for cleaning.

Key features

- Ability to rinse and disinfect up to eight EndoWrist instruments (two full operation sets + two spare instruments).
 - Possibility to close off unused connections to prevent loss of water pressure.
-

References

Documentation

The following documentation is available for the product and can be downloaded from Getinge.

Documentation	Contents	Paper	Electronic media	Available languages
Manual	Combined manual, including: - instructions for daily use. - instructions for assembly, installation and commissioning. - instructions for service and preventive maintenance.	✓	-	English.
Spare Parts	List of available spare parts. Download from Getinge SSU.	-	-	English.

- Not included at delivery of accessory ✓ Included at delivery of accessory

Accessories

Article No/Name	Description
6001567602	da Vinci Si Rack
6001248502	da Vinci Xi Rack
503185000	4-level wash cart for Getinge S-8668, S-8668T, 88 Turbo
Getinge S-8668, S-8668T, 88 Turbo	For ordering information, see separate Product specification for each product.

Getinge da Vinci compatibility

Getinge 86-series (S-8668 and S-8668T)

The Getinge 86-series consists of fully automatic, microprocessor controlled washer-disinfectors with high throughput. Their advanced automation capabilities provide options for automated loading and/or unloading that minimizes the need of staff intervention and shortens the waiting times, thus increasing the productivity. It is developed for cleaning and disinfecting at a thermally low or intermediate level, and for drying surgical instruments and other equipment. For more information about the washer-disinfectors, see separate product specifications for the Getinge 86-series.



Getinge 88-series (88 Turbo)

The Getinge 88 Turbo is a fully automatic, microprocessor controlled washer-disinfector with a capacity of 15 DIN trays and an extremely short program time. The Getinge 88 Turbo is developed for washing, intermediate level of disinfection and drying of surgical instruments (rigid and tubular), anesthetic accessories and other equipment. For more information about the washer-disinfectors, see separate product specifications for the Getinge 88-series.

da Vinci Rack

The da Vinci Rack for 4-level wash carts is specially designed for da Vinci instruments. It holds up to eight instruments, which equals two full operation sets including two spares. If less than eight instruments are loaded onto the rack, unused connections can be closed manually. Opening and closing is done without tools.

Note! The da Vinci Rack requires a 4-level wash cart (ordered separately).



Disinfection program

Depending on the type of washer-disinfector used, consult your Getinge representative for guidance regarding the **Disinfection** program.

86-series (S-8668/8668T)

The **Disinfection** program, can be used for thermal disinfection.

Program phases:

Post rinse 1, with HW, for 1 min (optional)

Post rinse 2, with HW, for 1 min (optional)

Final rinse, with RO, 194°F for 1 min

Drying, 1 min (adapt the time to meet local requirements)

88-series (88-5)

The **Disinfection** program is not included as default but required to be implemented before the first use of the da Vinci Rack.

Recommended settings:

Program phases:

Post rinse 1, with HW, 1 min (optional)

Post rinse 2, with HW, 1 min (optional)

Final rinse, with RO, 194°F, 1 min

Drying, 1-X min (adapt the time to meet local requirements)

Note! The **Disinfection** program is password protected (no cleaning is included).
(optional) = Not included in the standard program.



Getinge is a global provider of innovative solutions for operating rooms, intensive care units, sterilization departments and for life science companies and institutions. Based on our firsthand experience and close partnerships with clinical experts, healthcare professionals and medtech specialists, we are improving the everyday life for people – today and tomorrow.

Legal Manufacturer:

Getinge Disinfection AB · Ljungadalsgatan 11 · 352 46 Växjö · SWEDEN · +46 10 335 98 00

Getinge Infection Control AB · PO Box 69 · SE-305 05 Getinge · Sweden · +46 10 335 00 00 · info@getinge.com

www.getinge.com