

Getinge Clean

Chamber Cleaner Schedule 7

Chamber Cleaner Schedule 7 is a multipurpose, acidic cleaner/descaler. This product combines organic and inorganic acids with controlled foaming surfactant, inhibitors, rinse aids and penetrants for superior performance.



Application

For use in removing water scale and mineral deposits from sterilizers, utensil washers and cart washers. It can also be used on carts and other metal objects that have become discolored. Not for use on instruments.

Key Features

- Concentrated
- Aggressive acid quickly dissolves scale
- For moderate scale or very regular descaling (every 7 days)

Usage Information

This product should be used at a dose of 2–16 oz/gal (16–125 mL/L). Before applying this product, allow chamber to cool so that it is no more than warm to the touch. This will eliminate fuming or flash evaporation of the product solution. Dilute product with warm water as directed. Apply to cooled surface by spraying, wiping or brushing. Allow up to 5 minutes dwell time on surface, then thoroughly rinse with water.

- Do not add bleach, detergents, or any other chemical to this product. Added chemical may destroy the effectiveness of this cleaner.

NOTE: Some plastics and rubbers are not compatible with detergents or high water temperatures. Check with manufacturer for suitability.

Quality Statement: Confidence in Getinge is the most important quality criteria. This is the hallmark of all our external and internal commitments, activities and products. Products and services supplied by Getinge conform to the agreed terms and expectations. The achievement of these quality goals is the basis for continued competitive and successful enterprise.

Physical and Chemical Properties

- Clear, colorless, thin liquid
- Characteristic fragrance
- pH value: <1.0–1.5

NOTICE: For institutional and industrial use only. Not for home use. Refer to SDS for more information regarding safety, handling and storage and waste disposal methods.

Ordering Information

Getinge Clean

Order Number

4 x 1 Gallon

61301605287

Getinge is a registered trademark of Getinge AB, its subsidiaries, or affiliates in the United States or other countries • Copyright 2017 Getinge AB or its subsidiaries or affiliates • All rights reserved. • MCV00044184 REVA

Manufacturer: Intercon Chemical Co, 1100 Central Industrial Dr., St. Louis, MO 63110
Sales Office: Getinge, 1777 East Henrietta Road, Rochester, NY 14623

www.getinge.com

GETINGE 