

Getinge Clean

Descaler

Descaler is a noncorrosive, non flammable, scale remover fortified with wetting and penetrating agents.



Application

For use in the rapid removal of water scale, lime and rust deposits found on the water side of any equipment that is water cooled, water heated or water operated.

Key Features

- Improves efficiency, saves time and money, conserves energy and extends the useful life of equipment by keeping water passages free of mineral deposits
- Each gallon of Descaler used will dissolve approximately two pounds of water-formed deposits, from any type of water-operated equipment, in a matter of hours
- Free rinsing with water
- No unpleasant odor
- Fully effective at room temperature

Quality Statement: Confidence in Getinge is the most important quality criteria. This is the hallmark of all our external and internal commitments, activities and products. Products and services supplied by Getinge conform to the agreed terms and expectations. The achievement of these quality goals is the basis for continued competitive and successful enterprise.

Usage Information

Most Descaler cleaning can be accomplished in an average of two hours. Before applying Descaler, allow the boiler to cool so it is warm to the touch. This product may be used full strength or diluted with up to 5 parts water.

- Apply solution to surface by spraying, wiping or brushing.
- Allow to remain on surface for up to 5 minutes, then rinse with potable water.
- DO NOT allow this product to remain on metal surfaces for more than 20 minutes or discoloration, tarnishing and corrosive pitting may occur.
- Do not heat. Use at ambient temperature.
- Do not pressurize circulating solution or elevate temperature above 180°F (82°C).
- Do not circulate material for more than a four-hour period without consulting the manufacturer.

- Do not add bleach, detergents, or any other chemical to this product. Added chemical may destroy the effectiveness of this cleaner.
- Please refer to manufacturer's instructions.

NOTE: Some plastics and rubbers are not compatible with detergents or high water temperatures. Check with manufacturer for suitability.

Physical and Chemical Properties

- Clear liquid
- Hydrochloric fragrance
- pH value: <1.0 to 2.0

NOTICE: For institutional and industrial use only. Not for home use. Refer to SDS for more information regarding safety, handling and storage and waste disposal methods.

Ordering Information

Getinge Clean

4 x 1 Gallon

Order Number

61301605285

Getinge is a registered trademark of Getinge AB, its subsidiaries, or affiliates in the United States or other countries • Copyright 2017 Getinge AB or its subsidiaries or affiliates • All rights reserved. • MCV00044186 REVA

Manufacturer: Intercon Chemical Co, 1100 Central Industrial Dr., St. Louis, MO 63110

Sales Office: Getinge, 1777 East Henrietta Road, Rochester, NY 14623

www.getinge.com

GETINGE 