

# Getinge Clean

## Renuzyme WR

Renuzyme WR is a wide range, enzymatic, instrument detergent that contains both protease and lipase enzymes to digest a wide range of organic residues commonly found on surgical instruments.



### Application

For removal of proteins and fats from instruments and medical devices. For use in sinks, ultrasonic baths, cart washers, instrument washers and washer-disinfectors.

### Key Features

- Quickly and effectively removes blood, tissue, mucous, fats and other soils
- Neutral pH detergent
- Will not stain instruments or equipment made of glass, metal, plastic or rubber
- Effective in hard or soft water
- Can be used in both cold and warm water

**Quality Statement:** Confidence in Getinge is the most important quality criteria. This is the hallmark of all our external and internal commitments, activities and products. Products and services supplied by Getinge conform to the agreed terms and expectations. The achievement of these quality goals is the basis for continued competitive and successful enterprise.

## Usage Information

### Manual cleaning:

Use 1–2 oz/gallon (8–16 ml/L) of water. Discard soak solution when visibly soiled.

### Automatic Cleaning:

Inject 0.2–2 oz/gallon (2–16 ml/L) of water in prewash cycle. Always follow manufacturer's directions for the particular automatic washer being used. Amount of product used will vary depending on water quality. This product is most effective between the temperatures of 70°–140°F (21°–60°C). If you have additional questions, consult your Getinge representative.

- Do not add bleach, detergents or any other chemical to this product. Added chemical may destroy the effectiveness of this cleaner.

- Some plastics and rubbers are not compatible with detergents or high water temperatures. Check with manufacturer for suitability.

## Physical and Chemical Properties

- Clear, purple, thin liquid
- Sassafras fragrance
- pH value: 7.0–9.5

NOTICE: For institutional and industrial use only. Not for home use. Refer to Safety Data Sheet (SDS) for more information regarding safety, handling, storage and waste disposal methods.

---

## Ordering Information

### Getinge Clean

### Order Number

4 x 1 Gallon

61301605264

5 Gallons

61301605265

15 Gallons

61301605266

30 Gallons

61301605267

---

Getinge is a registered trademark of Getinge AB, its subsidiaries, or affiliates in the United States or other countries • Copyright 2017 Getinge AB or its subsidiaries or affiliates • All rights reserved. • MCV00044136 REVA

**Manufacturer:** Intercon Chemical Co, 1100 Central Industrial Dr., St. Louis, MO 63110

**Sales Office:** Getinge, 1777 East Henrietta Road, Rochester, NY 14623

[www.getinge.com](http://www.getinge.com)

**GETINGE** 